

IVALON[®]

Surgical Products
by fabco[®]

Nasal Packing

CE 0459

EC REP

MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany

fabco[®]

Manufactured for:

First Aid Bandage Company • 3 State Pier Road • New London, CT 06320 • USA
Phone: (860) 443-8499 • Fax: (860) 442-8699 • www.fabco.net

Medsorb Dominicana, S.A. • Zona Franca Industrial • San Pedro de Macoris • Dominican Republic

IVALON[®] is a registered trademark of Medsorb Dominicana S.A.

1391-50094-00117 Rev. F

Nasal Packing

(For use following intranasal surgery)

INSTRUCTIONS FOR USE

Step 1. BEFORE INSERTION

Coat the sides and leading edge of the epistaxis packing with a medical lubricant to ease insertion. IVALON[®] epistaxis packing may be trimmed with scissors if required.



Step 2. INSERTION

Grasp the nasal packing with forceps and gently insert along the nasal floor until the entire packing is within the nose.



Step 3. AFTER INSERTION

If necessary, hydrate the outer edge of the packing using a syringe containing saline or sterile water. Administer until the packing begins to expand (approximately 5cc for the 4.5 cm nasal packing and 10cc for the 8 cm nasal packing).

CAUTION

Tape string to cheek to prevent packing migration.

Step 4. BEFORE REMOVAL

Rehydrate the packing using a syringe with saline or sterile water (approximately 5cc for the 4.5 cm nasal packing and 10cc for the 8 cm nasal packing). Administer until the packing is fully saturated. Allowing the packing to remain in place for 5-10 minutes after rehydration facilitates removal. If an airway tube is present, remove it first, then rehydrate.



Step 5. REMOVAL

Use forceps to grasp the packing and gently withdraw.

CAUTION

- Sterile. Single use only. Re-use can result in microbial contamination causing subsequent health deterioration of patient. FABCO will not be responsible for any product that has been re-sterilized and/or re-used.
- Packing too tightly and/or for too long may result in necrosis.
- Do not cut sponges in close proximity to the surgical procedure. Fragments may enter the patient's wound site. Failure to remove sponge fragments from the patient may result in foreign body reaction.
- Always rehydrate before removal. Failure to re-hydrate may result in increased discomfort of the patient during packing removal.
- Removal must be conducted by a physician.
- Do not leave packing in place longer than three (3) days. Leaving packings in place too long may result in necrosis or increased risk for toxic shock (see below).
- Do not use if packet is opened, damaged or beyond expiration date to prevent risk of contamination and loss of sterility. Use proper aseptic techniques in all phases of handling.
- Used packings should be considered as bio waste and be disposed of in accordance with local regulations.

Toxic Shock

In rare instances, the physicochemical conditions associated with nasal/sinus surgery, both with and without nasal packings, may present a risk of toxic shock syndrome (TSS). Warning signs of TSS include: sudden fever (usually 102°F or more), vomiting, diarrhea, dizziness, fainting (or near fainting when standing up), and/or a rash that looks like a sunburn. If these or other signs of TSS appear, remove the packing at once and seek medical attention immediately. IVALON[®] sinus packings exhibit no antimicrobial properties, they are not bacteriostatic towards pre-existing infections, nor do they prevent the occurrence of new infections. In the case of pre-existing infections, it is recommended that appropriate treatment be instituted.

REF

Reorder number

LOT

Lot number

STERILE R

Sterilized by irradiation

△

Use Insert

⊘

Do not reuse

⌚

Expiration date

IVALON[®] is a registered trademark of Medsorb Dominicana S.A.

1391-50094-00117 Rev. F [ENG]